

CSM-PM100 QUICK SETUP GUIDE

NOTE: Consult the "Directions For Use" included with monitor for complete instructions, troubleshooting, and safety information

Setup



- 1 Remove the plastic film from face of the monitor, and install batteries.



- 2 Adjust the volume control in the battery compartment, close the cover, and tighten the screw



- 3 Attach the Velcro strap.

Use



- 1 Confirm that the green "IN USE" light is flashing.




- 2 **TEST THE ALARM:** Remove the magnetic dome from the socket. The alarm will sound, and the red "ALARM" light will flash.



- 3 Secure the monitor to back of the patient's chair or bed.



- 4 Attach the linen clip to the patient's clothing (out of the patient's reach)

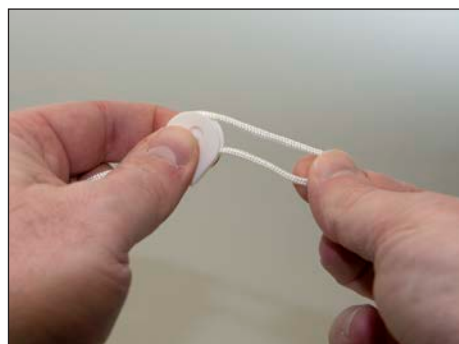

- 5 Adjust the string to a length that will pull out the magnetic dome to activate the alarm if the patient attempts an exit.



- 6 **CONNECTING TO NURSE CALL:** Plug a Nurse Call Connection Cable into the "NURSE CALL" socket on the monitor, connect the opposite end of cable into Nurse Call Station (an adaptor may be required) and test.

CSM-PM100 TROUBLESHOOTING CHECKLIST



This checklist has been developed to help identify common issues that can be quickly addressed to prevent unnecessary product returns. Please also refer to the product's Directions for Use and instructional video on Curbell's website (www.curbellmedical.com). If, after completing the checklist, the product continues to malfunction, please call your Curbell Representative at 888-222-2188.



No power/green "IN USE" light does not blink

- Batteries are installed according to the diagram in the battery compartment.
- Batteries are fresh and/or known to be good.

No alert is given to the facility's nurse call system

- The connection between the monitor and the nurse call patient station is securely connected and not damaged (a 1/4" mono nurse call jack and/or an adaptor for compatibility is required).

False alarm occurs

- Check that the magnetic dome is secured in the monitor socket.
- Check that the string length is not too short.

A chirping sound occurs

- Check for a Low battery LED indicator and replace the batteries if necessary.

No alarm sound occurs

- The volume level has been turned up.